



nt-trading GmbH & Co. KG

G.-Braun-Straße 18  
76187 Karlsruhe  
Germany

# NT-IQ implant library

for


# exocad®

Download and installation of the NT-IQ implant library for exocad DentalCAD.

The NT-IQ library contains three parts:

- the NT-IQ\_Preform library for one-piece titanium abutments and
- the NT-IQ\_TiB\_2-CON library for hybrid abutments on titanium bases and the 2CONnect system.
- the NT-IQ\_DIM library for digital implant models

- I) Save the file on an preferred local spot on your computer and copy the downloaded and compressed file „NT-IQ\_version.zip“
- II) Open the folder “Exocad-DentalCAD / DentalCAD App / library / implant“ in the Explorer. You find it on the drive, where your exocad software is installed
- III) Please pay attention: For the installation of the NT-IQ\_DIM Library open the folder „Exocad-DentalCAD / DentalCAD App / library / modelcreator / implants“

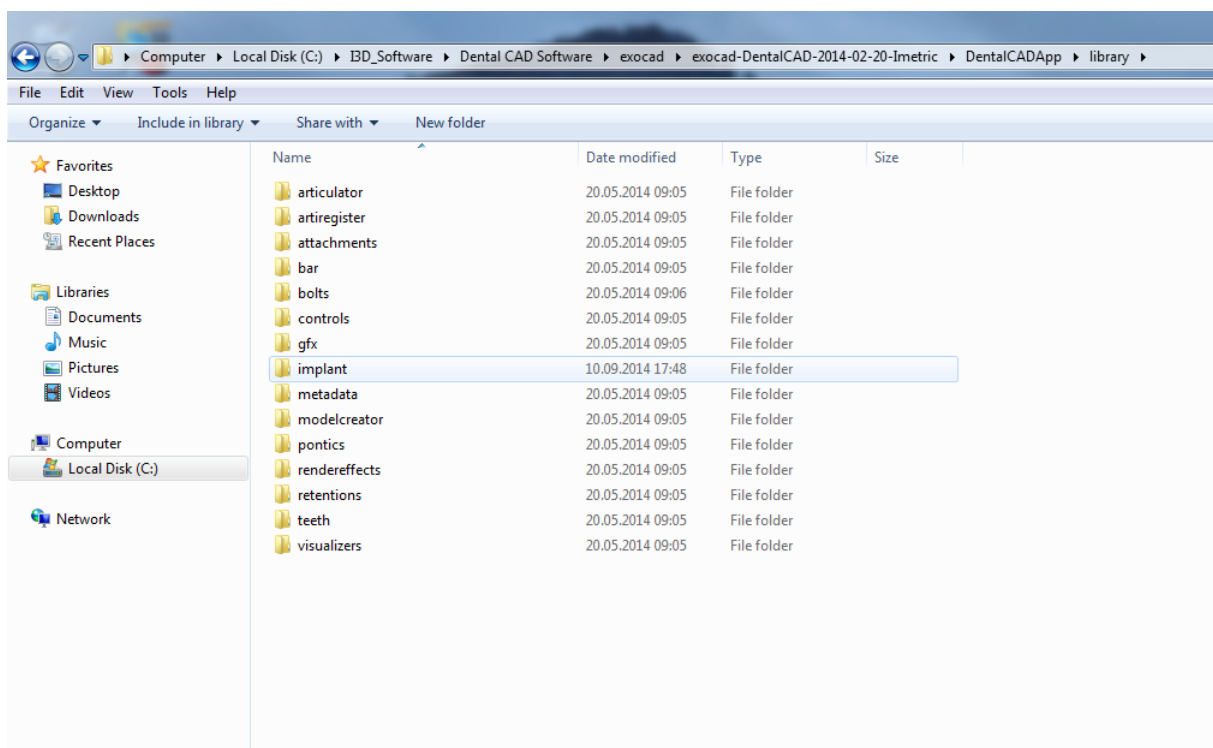



nt-trading GmbH & Co. KG

G.-Braun-Straße 18  
76187 Karlsruhe  
Germany


# NT-IQ implant library

for



Download and installation of the NT-IQ implant library for exocad DentalCAD

IV) Paste the copied file, open the folder „implant“ and unzip it

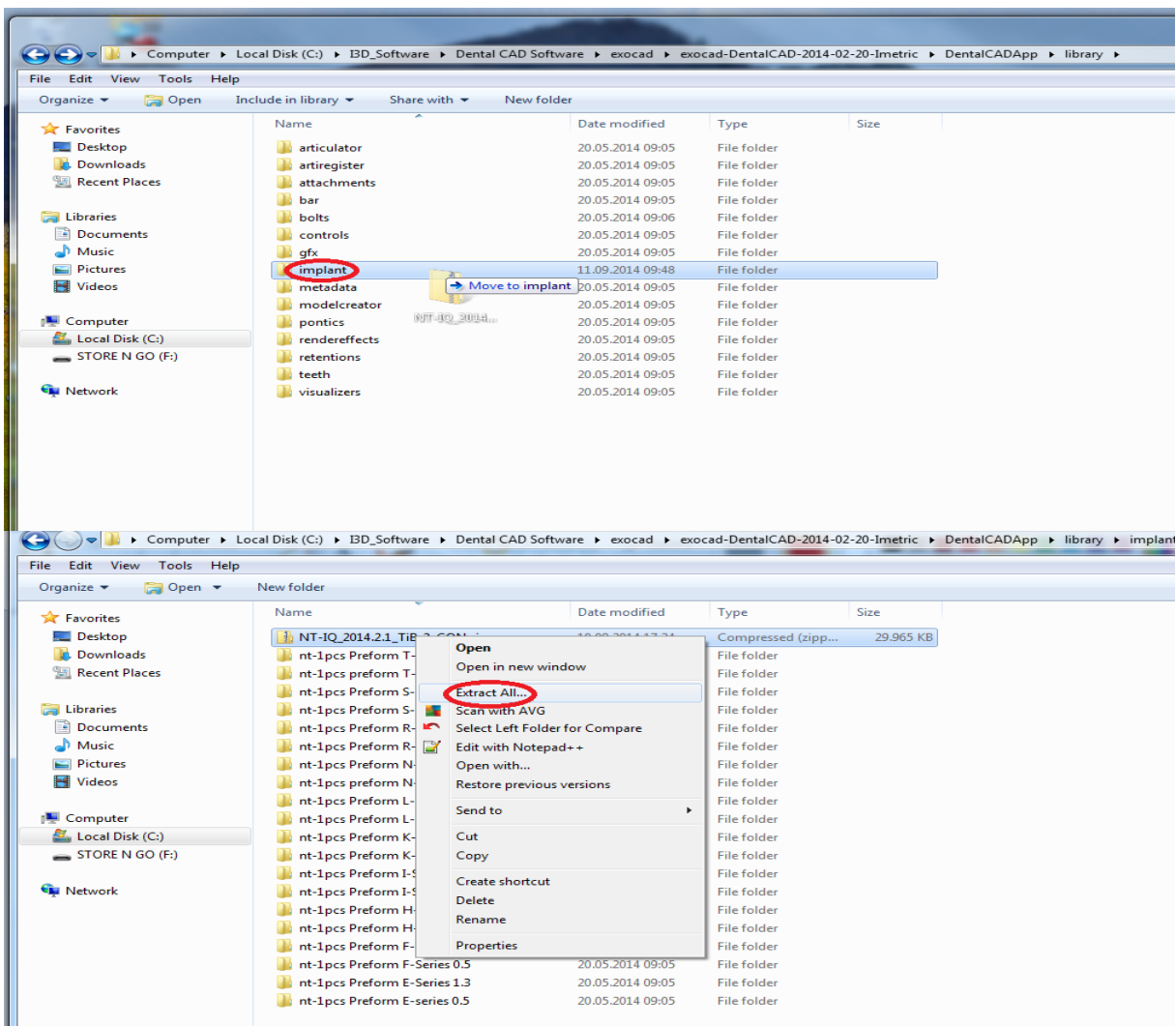



nt-trading GmbH & Co. KG

G.-Braun-Straße 18  
76187 Karlsruhe  
Germany


# NT-IQ implant library

for



Download and installation of the NT-IQ implant library for exocad DentalCAD

- V) After decompression you are able to work with our library. Please delete the .zip file from the folder „implant“

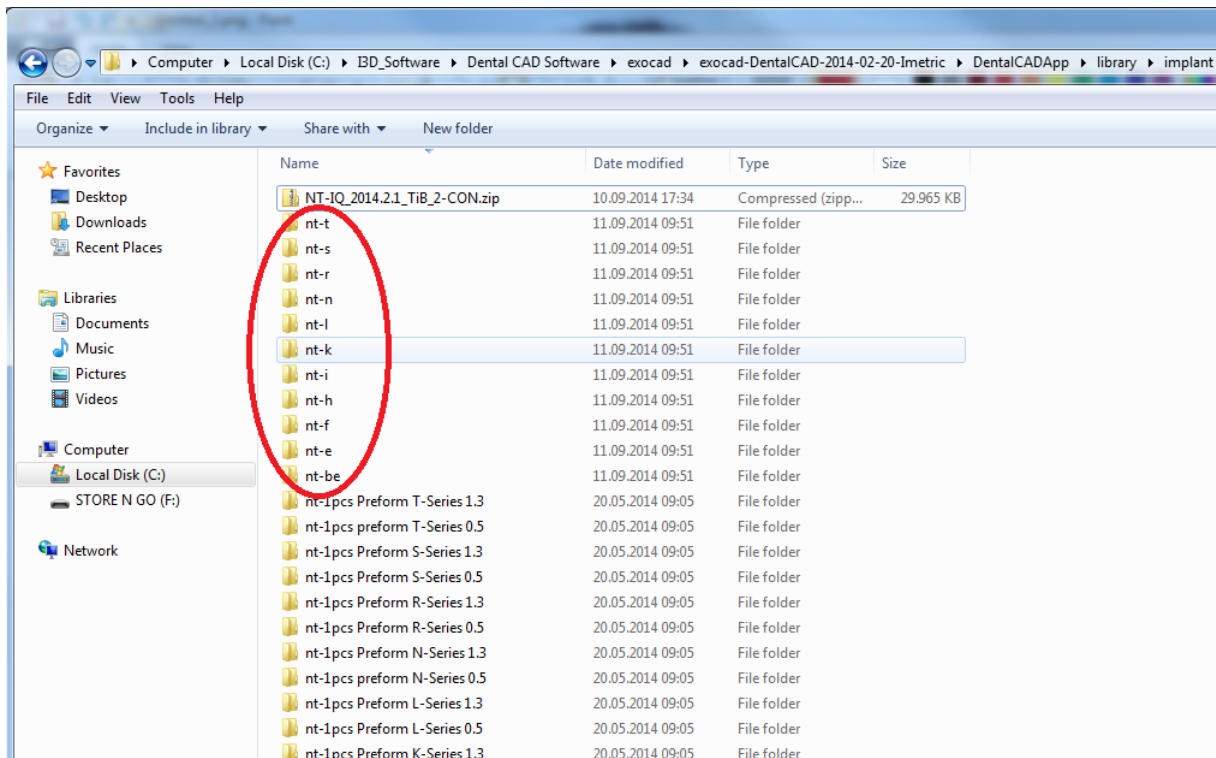



nt-trading GmbH & Co. KG

G.-Braun-Straße 18  
76187 Karlsruhe  
Germany

# NT-IQ implant library

for

# exocad®

Download and installation of the NT-IQ implant library for exocad DentalCAD

VI) Restart the exocad software

