


# nt-iQ implant library for 3shape

Download and installation of the nt-iQ implant library for 3shape Dental Designer.

- 1) Please press the icon „nt-iQ library 3shape“ in our download area

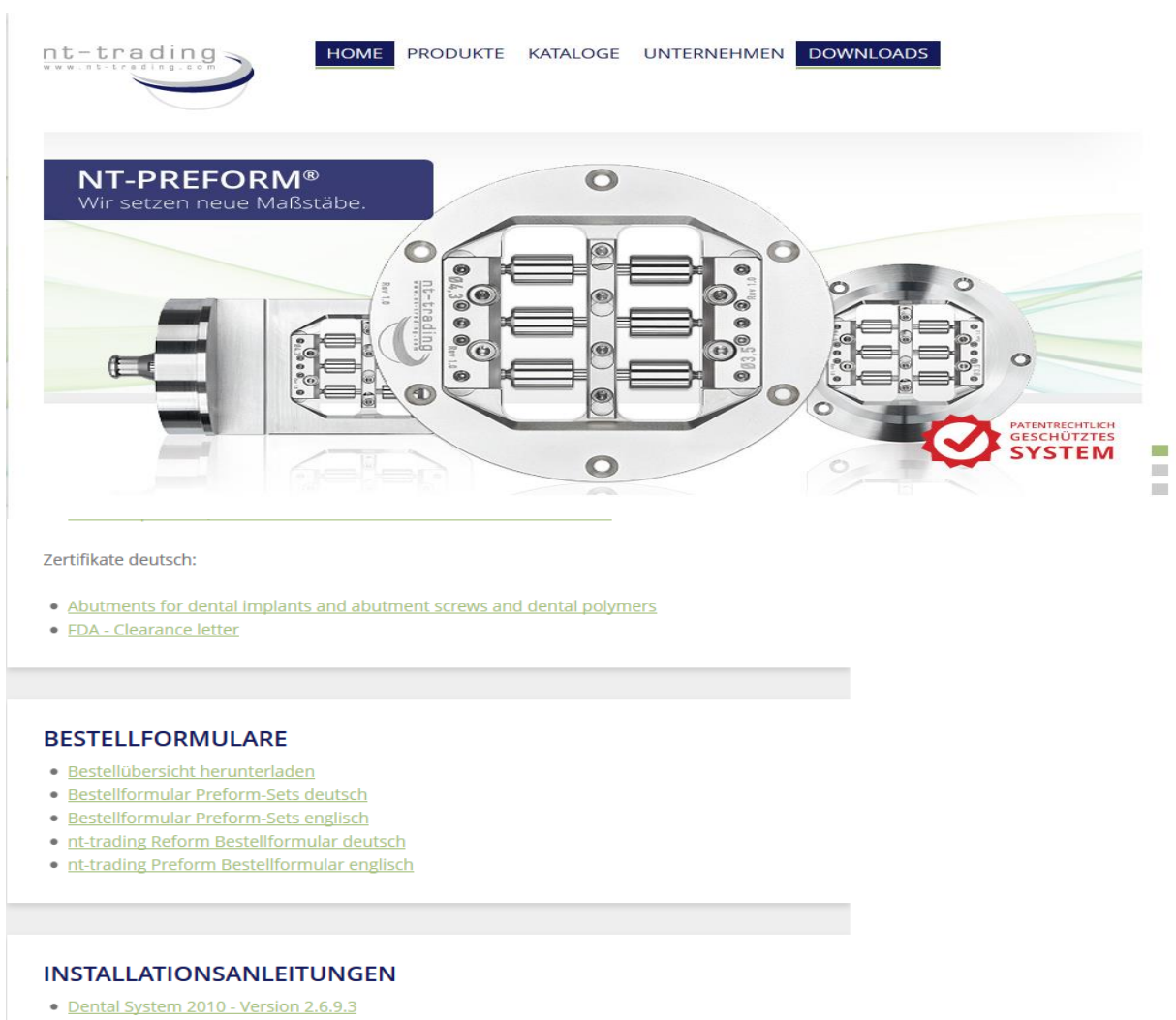

nt-trading  
www.nt-trading.com

HOME PRODUKTE KATALOGE UNTERNEHMEN DOWNLOADS

**NT-PREFORM®**  
Wir setzen neue Maßstäbe.

PATENTRECHTLICH  
GESCHÜTZTES  
SYSTEM

Zertifikate deutsch:

- [Abutments for dental implants and abutment screws and dental polymers](#)
- [FDA - Clearance letter](#)

**BESTELLFORMULARE**

- [Bestellübersicht herunterladen](#)
- [Bestellformular Preform-Sets deutsch](#)
- [Bestellformular Preform-Sets englisch](#)
- [nt-trading Reform Bestellformular deutsch](#)
- [nt-trading Preform Bestellformular englisch](#)


**INSTALLATIONSANLEITUNGEN**

- [Dental System 2010 - Version 2.6.9.3](#)

# nt-iQ implant library for 3shape

Download and installation of the nt-iQ implant library for 3shape Dental Designer.


- II) The download of the zipped folder “NT\_IQ\_2013.1.1.zip” starts automatically
- III) Save the folder “NT\_IQ\_2013.1.1.zip” on an preferred local spot on your computer
- IV) Unzip the folder “NT\_IQ\_2013.1.1.zip”, it contains the file “NT\_IQ\_2013.1.1.dme”
- V) Start the 3shape “Control Panel”
- VI) Go to “Tools/Import/Export”

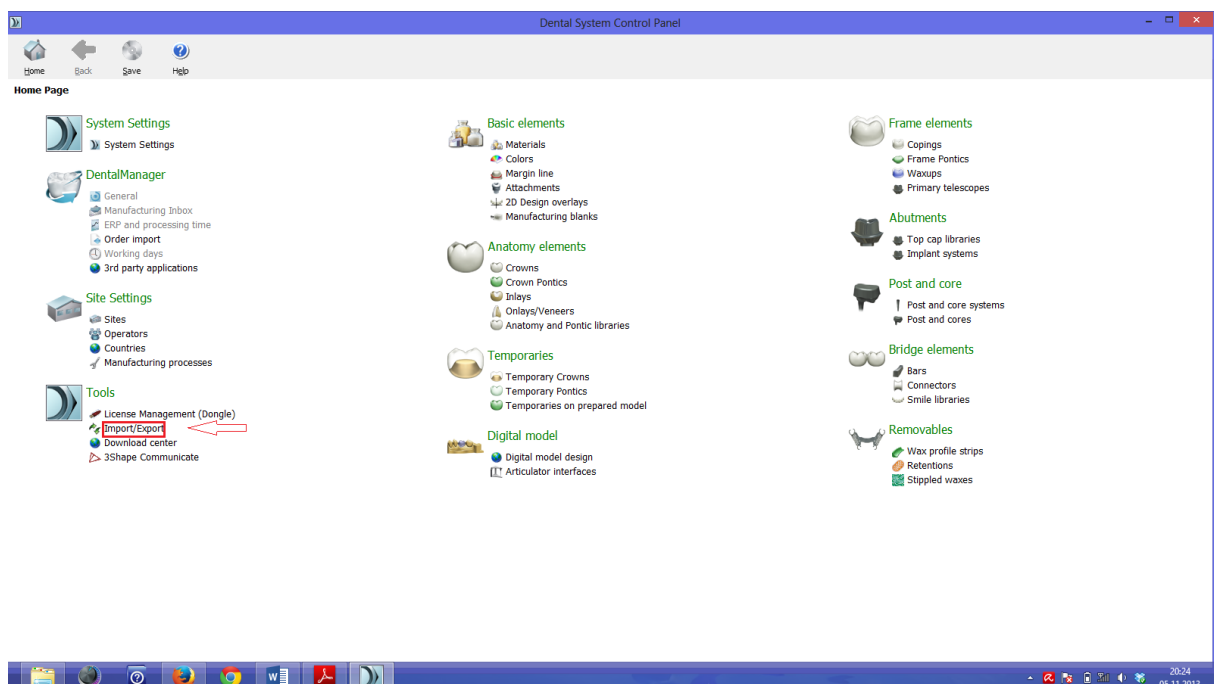

# nt-iQ implant library for 3shape

Download and installation of the nt-iQ implant library for 3shape Dental Designer.

VII) Start the button "Import materials"

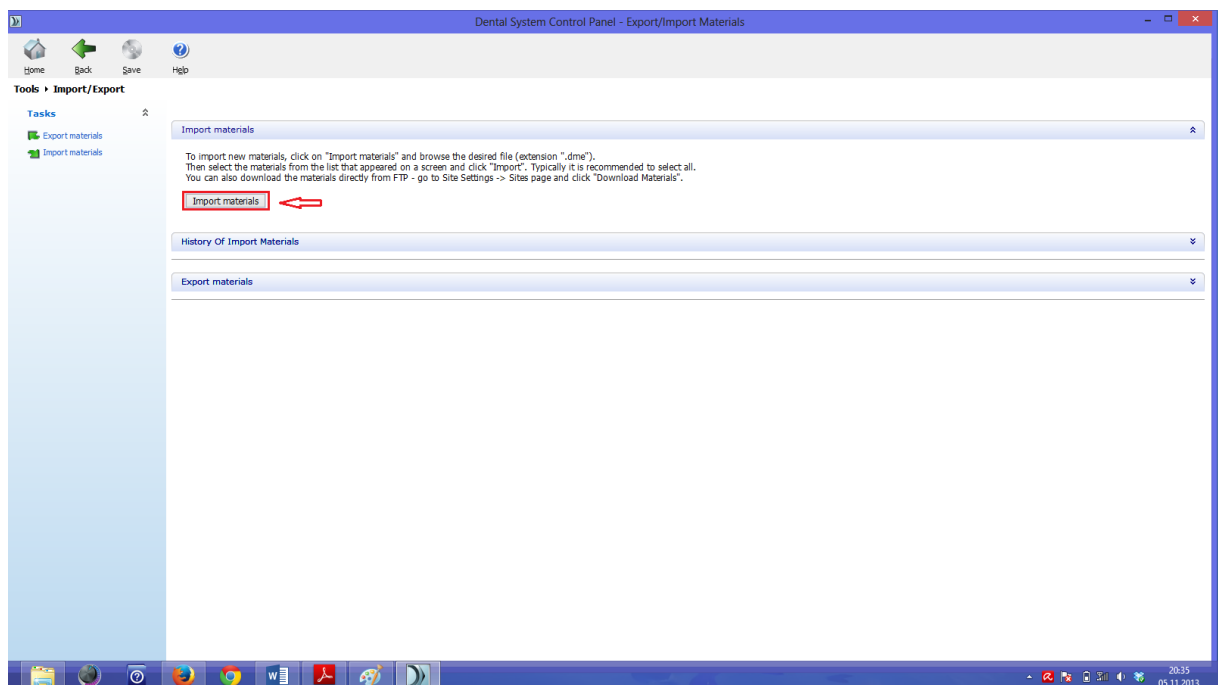




nt-trading GmbH & Co. KG

G.-Braun-Straße 18  
76187 Karlsruhe

# nt-iQ implant library for 3shape

Download and installation of the nt-iQ implant library for 3shape Dental Designer.

VIII) Windows Explorer opens and you are able to import the downloaded file  
“NT\_IQ\_2013.1.1.dme“

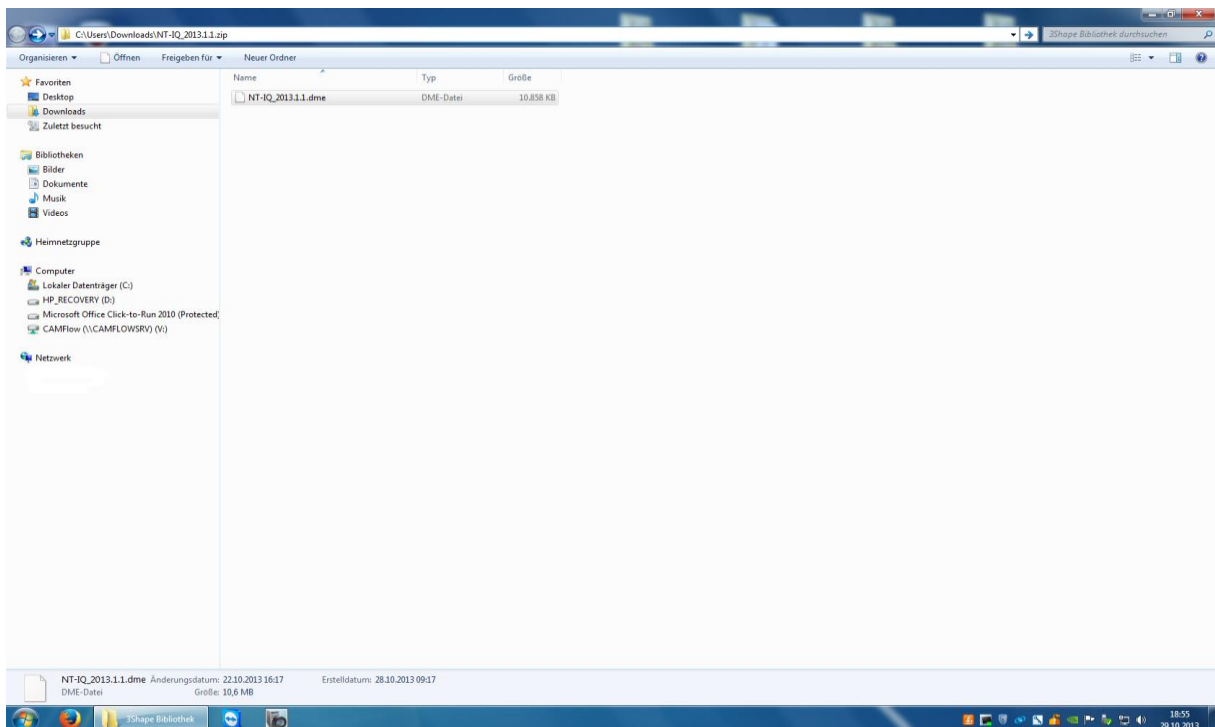



nt-trading GmbH & Co. KG

G.-Braun-Straße 18  
76187 Karlsruhe

# nt-iQ implant library for 3shape

Download and installation of the nt-iQ implant library for 3shape Dental Designer.

II) Our library is available

