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## INSTRUCTION FOR USE [ENG] NT X-TRA HOLDER

### 1. PRODUCT DESCRIPTION

By means of inserts, the multi-part NT X-tra holder offers a clamping option for NT Milling Analogs where an abutment is fixed in place for manual milling.

### 2. FUNCTION OVERVIEW


- Solid stainless steel grip for optimal heat dissipation during milling
- Suitable for more than 26 different implant systems
- 3 inserts allow the use of all NT Milling Analogs for 26 implant systems
- Ring marking of inserts enables clear assignment of abutments (Preform holder set)
  - 1 ring / Milling Analog, abutment grip, yellow / 3.5 mm
  - 2 rings / Milling Analog, abutment grip, red / 4.3 mm
  - 3 rings / Milling Analog, abutment grip, blue / 5.0 mm
- Easy clamping of the inserts and fixing of the analogs with a conical screwdriver

Solid stainless steel grip for holder fixture of Inserts 1-3

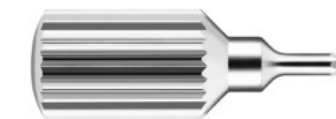Inserts 1-3 for holder fixture of the NT Milling Analogs

Conical screwdriver for X-tra holder and inserts

### 3. INITIAL USE

Select the appropriate insert for the diameter of the NT Milling Analog to be clamped and insert it into the holder fixture of the X-tra holder. Tighten the screw with a screwdriver.

Align the NT Milling Analog, retention surface in the direction of the screw, insert it into the insert holder fixture as far as it will go, and fix it in place with a screw.

The milling of the abutment, fixed in place in the Analog with a screw, is carried out in a suitable environment by dental implantology specialists in accordance with the specifications of the particular work instructions.

### 4. CARE INSTRUCTIONS

The cleaning of the individual components should be performed separately and carefully after each use. Components that are compromised by damage must be excluded from further use and replaced by new components.

### 5. SAFETY

Before using it, the user is obliged to independently check the product as to its suitability and use for the intended purpose.

The use of the NT X-tra holder is the responsibility of the user. Improper use of the X-tra holder can result in injury. For any damages that are caused, a contributory negligence on the part of the user will result in a reduction or the total exclusion of any liability on the part of nt-trading GmbH & Co. KG. This is particularly the case if the user instructions or warnings are ignored, or if the user inadvertently misuses the device.

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