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### nt-Preforms

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## Instruction for use [ENG]

### nt-Prefoms

#### Compatibility of the series to the implant systems

Serien (nt-Preform) / Series (nt-Preform)	Kompatibles Implantatsystem / Compatible Implant System
BEG	BEGO Implant Systems®/Semados® S, RS-RX, RI-Line
BIO	BioHorizons® / internal
BPS	BEGO Implant Systems/Semados® SC/SCX-/ RS/RSX-/RI-Line mit PS Design
BRE	Bredent Medical® / Sky®
CAM	Altatec® / Camlog®
COL	Altatec® / Conelog®
E	Nobel Biocare™ / Replace Select®
F	Nobel Biocare™ / Nobel Active™
H	Biomet 3i® / Certain®
HIO	HIOSSEN® / OSSTEM TS®
I	Biomet 3i® / Osseotite® External Hex®
K	Nobel Biocare™ / Brånemark®
L	Straumann / Bone Level®
MEG	MegaGen/ AnyRidge®
MEX	Medentis Medical® / ICX®
MIC	MIC® / C1®
MIS	MIS® / SEVEN®
N	Straumann / SynOcta®
NEO	NEOSS Implant System® / NEOSS™
R	Zimmer Dental / Tapered Screw-Vent®
S	DENTSPLY Implants® / Astra Tech Implant System OsseoSpeed® TX
SEV	DENTSPLY Implants® / Astra Tech Implant System™ EV
T	DENTSPLY Implants® / Xive®
THO	Thommen Medical® / SPI®

UPR	Sweden & Martina Implantology™ / Premium Kohno®
UTG	Sweden & Martina Implantology™ / Premium TG®
Y	DENTSPLY Implants® ANKYLOS®

Please consult the product catalogue of nt-trading for article numbers of the products for the respective diameters.

### Torques

Ncm	Serie									
15	THO (Ø 3,5mm)	Y								
20	H	COL	CAM	HIO (Ø 3,5mm)						
25	F (NP)	S	T	BRE	THO (Ø 4,0-6,0 mm)	UPR	UTG	SEV		
30	R	MEX	MIS	BEG	BIO	BPS	NEO	MIC	HIO (Ø 4,0-5,0 mm)	MEG
35	E	F (RP)	I	K	L	N				

Mechanical treatment of the connection part will damage the correct fitting of the products on the implant.

For fixation of the products on the implant, the correct torque force, recommended by the implant manufacturer, has to be considered carefully to avoid the damage of the implant-bone connection.

### NT-PREFORM

The nt-Preform is a cylinder made of Titanium Grade 5 with a pre-manufactured connection surface. It can be milled and shortened to create an individual customized abutment. The nt-Preform abutment is oversized in order to allow for the creation of a customized abutment through the milling process. By providing a customized abutment, each patient's aesthetic and functional demands can be met.

The nt-Preform abutment is secured in a holder by a milling screw (min. 10 Ncm). A corresponding lab analog, which is connected to the nt-Preform by the milling screw, ensures that no rotation can take place. When milling the nt-Preform, always ensure the proper milling burs are used along with adequate and continuous cooling. Inappropriate modification of the vertical axis angle, wall thickness (0,4 mm) and/or adjustment (min. 3 mm) of the nt-Preform abutment can lead to material defects or failure. It is very important to always use the appropriate holder for securing the nt-Preform. Always ensure there is compatibility and optimized settings between all system components including CAD/CAM for example: 3Shape, Exocad or Dental Wings nt-Preforms and the patient's data to guarantee optimum results for the completed dental prosthesis. The manufacturing of dental prosthetic products and the use of dental CAD/CAM systems must be carried out by qualified personnel only. nt-trading will not be held responsible for incorrectly designed or milled nt-Preform abutments.

### COMPOSITION:

nt-Preform: Ti6Al4V, medical grade 5, ASTM F136

## **General Information**

### **WARNING:**

Safety hint: metal dust is harmful to your health. When milling and sandblasting use a suction extraction system and a breathing mask.

### **SECONDARY EFFECTS:**

Allergies to the alloy or contents of the alloy or electrochemically based reactions may very rarely occur.

### **REACTIONS:**

In case of occlusal or approximal contact of different alloys electrochemically based reactions may very rarely occur.

### **WARRANTY:**

10 Years on the mechanical stability of the nt-trading Abutment, if it was processed according to the Instruction for use. Whether given verbally, in writing or by practical instructions, our recommendation for use is based upon own experience and trials and can only be considered as standard values. Our products are subject to a constant further development. Therefore alternations in construction and composition are reserved.

### **CLEANING, DISINFECTION and STERILIZATION:**

The listed products in this IFU are supplied in non-sterile condition. The components should be cleaned, disinfected and in specific clinical procedures and cases be sterilized, prior to use after they are received from the dental laboratory (no liability on disregard). Effective cleaning and disinfection is an indispensable requirement for effective sterilization of the medical device. Prior to sterilization, please keep medical devices clean when handling in the laboratory and operatory.

Additionally, please pay attention to legal regulations valid for your local areas as well as to the hygienic instructions of your dental practice. This applies particularly to the different guidelines regarding the inactivation of prions.

Please observe all manufacturers guidelines for disinfection and cleaning agents with special regard to the concentration, exposure time and temperature. Only pH-neutral disinfection solutions without chlorine, ammoniac and aldehydes and with a proven effectiveness against HBV, HCV, and HIV must be used. The products have to meet the respective national regulations for disinfectants. If disinfectants containing aldehydes are used, this might lead to a possible fixation of proteins. Please use only freshly prepared solutions.

#### **1. Pre-disinfection (avoidance of cross contaminations)**

Place the devices in a germicidal bath\* immediately after use. Remove all residues and disassemble demountable products.

#### **2. Cleaning**

Please use distilled water and neutral cleaning agents only. The internal irrigation tube has to be cleaned with a Miller needle and must be rinsed with distilled water at the beginning and end of the exposure time using a disposable syringe (min. 10 ml). The products must be cleaned with a plastic instrument cleaning brush and then rinsed with distilled water. Please control all products after cleaning in order to avoid either damaging or corrosion. Damaged products must be replaced.

### 3. Rinsing and Drying

After removal of the products from the germicidal bath, all components must be rinsed 3 times with distilled water /e.g. Aqua purificata). Please dry all components thoroughly with a lint-free disposable cloth. For the cleaning of the internal irrigation tube oil-free compressed air is mandatory. Please re-check all parts for damage or corrosion afterwards.

### 4. Disinfection

We recommend a high level disinfectant such as, Cidex OPA (Johnson & Johnson) for disinfection of the abutments and screws.

- a) Soak the medical device in the disinfectant solution for the required amount of time. See instructions for use of Cidex OPA.
- b) Remove the medical device from the disinfectant solution.
- c) Rinse at least three times with highly purified water.
- d) Air dry and package the devices immediately.

### 5. Sterilization:

If no sterilization device is available in the laboratory, this information should be forwarded to the dentist so proper sterilization can occur. Please use only validated sterilization procedures for the sterilization of the devices. Other sterilization procedures must not be used.

Reusability: You may only sterilize the medical device one time. In case of inadvertent contamination, you may re-sterilize one time after cleaning and disinfection.

### 6. Steam sterilization:

- Fractionated vacuum procedure or gravity procedure (with sufficient product drying)
- Steam sterilizer according to ISO 17665: 2006 or EN 13060 and EN 285 respectively or equivalent national standards
- Validated according to EN ISO/ANSI AAMI 17665 (in past: EN 554/ANSI AAMI ISO 11134) (valid IQ/OQ) (commissioning and product specific performance qualification)
- Sterilization time not less than 15 minutes at 121 ° C (250 °F) (listed exposure times are at sterilization temperature)

### 7. Storage

Store the sterilized parts dry and dust-free at room temperature (18°-25° C/64°-77° F).

**MR Conditional**

Application only by qualified personnel!

After removing the prosthesis, regardless of whether it is cemented or screwed, the abutment or the abutment screw must be retightened with the specified torque. See tables for torques in IFU.

**MANUFACTURER:**  
**nt-trading GmbH & Co. KG**  
G.-Braun-Str. 18  
76187 Karlsruhe, Germany

Fon: +49-721-91547160  
Fax: +49-721-91547161  
E-mail: info@nt-trading.com

**Products indicated with ® are registered brand names of the manufacturers**

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