

# NT



## **TITANIUM BASE NX**

*Titanium Base NX (NoHex) for implant supported hybrid bridges/-bars*

## NT-TRADING TIBASE NX LIBRARY

Non-engaged titanium bases (NoHex) for the production of prosthetic constructions on multiple implants (bars, bridges). These are available in 2 different gingival heights (GH) and can be customized in height from 5mm (CH5) to 3mm (CH3).



### **BENEFITS:**

- non-engaged
- cylinder height can be adjusted from 5 mm to 3 mm
- different gingiva heights available
- perfect adaption to the gingival margin by optimized transition zone between implant and abutment
- improved for CAD/CAM process



### **APPLICATION:**

- for individual hybrid bridges/ -bars on multi-implant cases made from ceramics (zirconium oxide, lithium disilicate) or synthetic (PMMA, PEEK, composite)



### **TECHNICAL DETAILS:**

- Titanium Grade 5 ELI

# TITANIUM BASE NX


## SELECTION OF TIBASE NX (NOHEX):

3shape


exocad




Select nt-trading TiB NX under "Category"

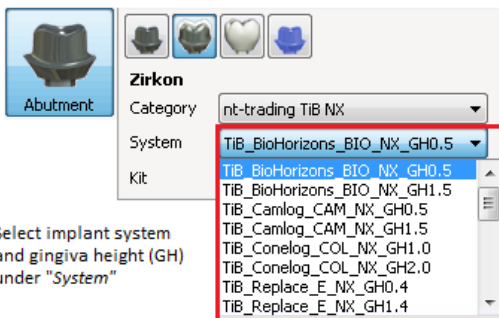


Select implant system and gingiva height (GH) under "System"

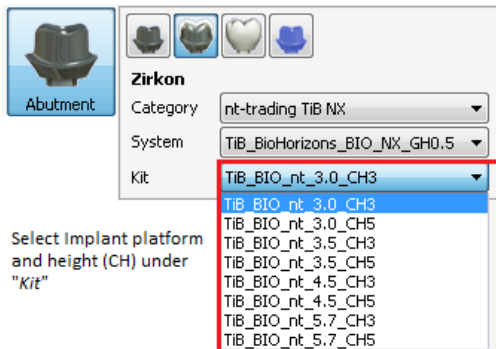


Select Implant platform and height (CH) under "Kit"

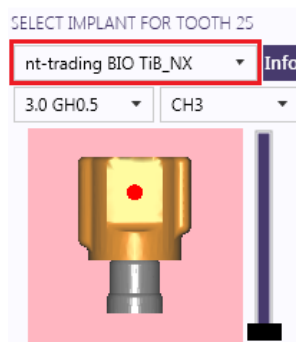

Select nt-trading TiB NX and implant system

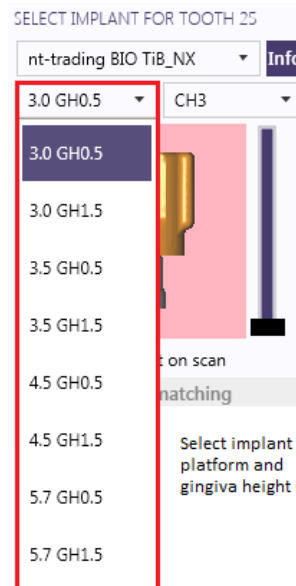



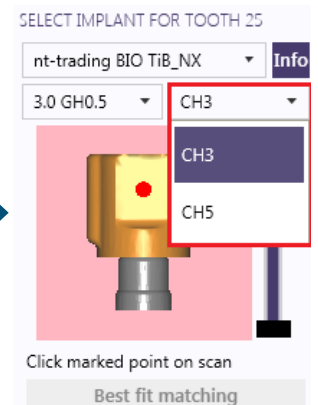



Select implant platform and gingiva height (GH)



Select height (CH)







## ITEM SELECTION (FAX TEMPLATE **+49 721 915471-61**):

SERIES	PLATFORM	1. GINGIVA-HEIGHT	ITEM NUMBER	2. GINGIVA HEIGHT	ITEM NUMBER	QUANTITY	
						1. GH	2. GH
<b>BIO-SERIES</b>							
	Ø 3.0	GH 0.5 mm	BIO 82300005	GH 1.5 mm	BIO 82300015		
	Ø 3.5	GH 0.5 mm	BIO 82350005	GH 1.5 mm	BIO 82350015		
	Ø 4.5	GH 0.5 mm	BIO 82450005	GH 1.5 mm	BIO 82450015		
	Ø 5.7	GH 0.5 mm	BIO 82570005	GH 1.5 mm	BIO 82570015		
<b>BRE-SERIES</b>							
	Ø 3.5	GH 0.5 mm	BRE 82350005	GH 1.5 mm	BRE 82350015		
	Ø 4.0	GH 0.5 mm	BRE 82400005	GH 1.5 mm	BRE 82400015		
<b>CAM-SERIES</b>							
	Ø 3.3	GH 0.5 mm	CAM 82330005	GH 1.5 mm	CAM 82330015		
	Ø 3.8	GH 0.5 mm	CAM 82380005	GH 1.5 mm	CAM 82380015		
	Ø 4.3	GH 0.5 mm	CAM 82430005	GH 1.5 mm	CAM 82430015		
	Ø 5.0	GH 0.5 mm	CAM 82500005	GH 1.5 mm	CAM 82500015		
<b>COL-SERIES</b>							
	Ø 3.3	GH 1.0 mm	COL 82330010	GH 2.0 mm	COL 82330020		
	Ø 3.8/4.3	GH 1.0 mm	COL 82384310	GH 2.0 mm	COL 82384320		
	Ø 5.0	GH 1.0 mm	COL 82500010	GH 2.0 mm	COL 82500020		
<b>E-SERIES</b>							
	Ø 3.5 NP	GH 0.4 mm	E 82350004	GH 1.4 mm	E 82350014		
	Ø 4.3 RP	GH 0.4 mm	E 82430004	GH 1.4 mm	E 82430014		
	Ø 5.0 WP	GH 0.4 mm	E 82500004	GH 1.4 mm	E 82500014		
	Ø 6.0	GH 0.4 mm	E 82600004	GH 1.4 mm	E 82600014		
<b>F-SERIES</b>							
	Ø 3.5 NP	GH 0.8 mm	F 82350008	GH 1.8 mm	F 82350018		
	Ø 4.3/5.0 RP	GH 0.8 mm	F 82435008	GH 1.8 mm	F 82435018		

fax form - please fill







# TITANIUM BASE NX

## ITEM SELECTION (FAX TEMPLATE +49 721 915471-61):

SERIES	PLATFORM	1. GINGIVA-HEIGHT	ITEM NUMBER	2. GINGIVA HEIGHT	ITEM NUMBER	QUANTITY 1. GH   2. GH	
<b>H-SERIES</b>							
	Ø 3.4	GH 0.3 mm	H 82340003	GH 1.3 mm	H 82340013		
	Ø 4.1	GH 0.3 mm	H 82410003	GH 1.3 mm	H 82410013		
	Ø 5.0	GH 0.3 mm	H 82500003	GH 1.3 mm	H 82500013		
<b>HIO-SERIES</b>							
	Ø 3.5 MC	GH 1.0 mm	HIO 82350010	GH 2.0 mm	HIO 82350020		
	Ø 4.0-6.0 RC	GH 1.0 mm	HIO 82406010	GH 2.0 mm	HIO 82406020		
<b>K-SERIES</b>							
	Ø 3.5 NP	GH 0.6 mm	K 82350006	GH 1.6 mm	K 82350016		
	Ø 4.1 RP	GH 0.5 mm	K 82410005	GH 1.5 mm	K 82410015		
	Ø 5.1 WP	GH 0.6 mm	K 82510006	GH 1.6 mm	K 82510016		
<b>L-SERIES</b>							
	Ø 3.3 NC	GH 0.7 mm	L 82330007	GH 1.7 mm	L 82330017		
	Ø 4.1/4.8 RC	GH 0.7 mm	L 82414807	GH 1.7 mm	L 82414817		
<b>MEG-SERIES</b>							
	Ø 3.5-5.5	GH 0.3 mm	MEG 82355503	GH 1.3 mm	MEG 82355513		
<b>MEX-SERIES</b>							
	Ø 3.7-4.8 STD	GH 1.0 mm	MEX 82414810	GH 2.0 mm	MEX 82414820		

fax form - please fill
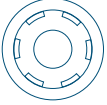
## ITEM SELECTION (FAX TEMPLATE +49 721 915471-61):

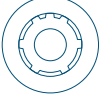SERIES	PLATFORM	1. GINGIVA-HEIGHT	ITEM NUMBER	2. GINGIVA HEIGHT	ITEM NUMBER	QUANTITY 1. GH   2. GH	
<b>MIS-SERIES</b>							
	Ø 3.3 NP	GH 0.5 mm	MIS 82330005	GH 1.5 mm	MIS 82330015		
	Ø 3.75/4.2 SP	GH 0.5 mm	MIS 82374205	GH 1.5 mm	MIS 82374215		
	Ø 5.0 WP	GH 0.5 mm	MIS 82500005	GH 1.5 mm	MIS 82500015		
<b>N-SERIES</b>							
	Ø 4.8 RN	GH 0.2 mm	N 82480002	-	-		-
	Ø 6.5 WN	GH 0.3 mm	N 82650003	-	-		-
<b>NEO-SERIES</b>							
	Ø 3.25	GH 0.5 mm	NEO 82320005	GH 1.5 mm	NEO 82320015		
	Ø 3.5-5.5	GH 0.5 mm	NEO 82355505	GH 1.5 mm	NEO 82355515		
<b>R-SERIES</b>							
	Ø 3.5	GH 0.5 mm	R 82350005	GH 1.5 mm	R 82350015		
	Ø 4.5	GH 0.5 mm	R 82450005	GH 1.5 mm	R 82450015		
	Ø 5.7	GH 0.5 mm	R 82570005	GH 1.5 mm	R 82570015		
<b>S-SERIES</b>							
	Ø 3.5/4.0	GH 0.8 mm	S 82354008	GH 1.8 mm	S 82354018		
	Ø 4.5/5.0	GH 0.8 mm	S 82455008	GH 1.8 mm	S 82455018		
<b>SEV-SERIES</b>							
	Ø 3.0	GH 1.0 mm	SEV 82300010	GH 2.0 mm	SEV 82300020		
	Ø 3.6	GH 1.0 mm	SEV 82360010	GH 2.0 mm	SEV 82360020		
	Ø 4.2	GH 1.0 mm	SEV 82420010	GH 2.0 mm	SEV 82420020		
	Ø 4.8	GH 1.0 mm	SEV 82480010	GH 2.0 mm	SEV 82480020		
	Ø 5.4	GH 1.0 mm	SEV 82540010	GH 2.0 mm	SEV 82540020		

fax form - please fill

# TITANIUM BASE NX

## ITEM SELECTION (FAX TEMPLATE +49 721 915471 -61):

SERIES	PLATFORM	1. GINGIVA-HEIGHT	ITEM NUMBER	2. GINGIVA HEIGHT	ITEM NUMBER	QUANTITY 1. GH   2. GH	
<b>T-SERIES</b>							
	Ø 3.4	GH 0.3 mm	T 82340003	GH 1.3 mm	T 82340013		
	Ø 3.8	GH 0.3 mm	T 82380003	GH 1.3 mm	T 82380013		
	Ø 4.5	GH 0.6 mm	T 82450006	GH 1.6 mm	T 82450016		
	Ø 5.5	GH 0.6 mm	T 82550006	GH 1.6 mm	T 82550016		
<b>Y-SERIES</b>							
	Ø 3.5 C/	GH 1.0 mm	Y 82350010	GH 2.0 mm	Y 82350020		

fax form - please fill

## COMPATIBILITY OF THE SERIES:

- BIO-Series compatible to BioHorizons® / internal
- BRE-Series compatible to Bredent Medical® / Sky®
- CAM-Series compatible to Altatec® / CAMLOG®
- COL-Series compatible to Altatec® / Conelog®
- E-Series compatible to NOBEL BIO CARE™ / REPLACE SELECT®
- F-Series compatible to NOBEL BIO CARE™ / NOBEL ACTIVE™
- H-Series compatible to BIOMET 3i® / CERTAIN®
- HIO-Series compatible to HIOSSEN® / OSSTEM TS®
- K-Series compatible to NOBEL BIO CARE™ / Brånemark®
- L-Series compatible to STRAUMANN / BONELEVEL®
- MEG-Series compatible to MEGAGEN / AnyRidge®
- MEX-Series compatible to Medentis Medical® / ICX®
- MIS-Series compatible to MIS® / SEVEN®
- NEO-Series compatible to NEOSS Implant System® / NEOSS™
- N-Series compatible to Straumann / SynOcta®
- R-Series compatible to ZIMMER Dental / TAPERED SCREW-VENT®
- S-Series compatible to DENTSPLY Implants® / Astra Tech Implant SystemOsseoSpeed® TX
- SEV-Series compatible to DENTSPLY Implants® / ASTRA TECH Implant System™ EV
- T-Series compatible to DENTSPLY Implants® / XIVE®
- Y-Series compatible to DENTSPLY Implants® / ANKYLOS®



**nt-trading GmbH & Co. KG**  
G.-Braun-Straße 18 · 76187 Karlsruhe · Germany

info@nt-trading.com · www.nt.dental  
Phone +49 (0) 721 - 915471 - 60 · Fax: +49 (0) 721 - 915471 - 61